POINT OF CONTACT ALSICO N.V. Zonnestraat 223 9600 Ronse - Belgium T + 32 55 23 71 71 F + 32 55 23 71 79	Gebruiksaanwijzing - Instructions d'utilisation - Gebrauchsanweisung - Users instructions HERBRUIKBAAR ALSICO MASK Performance - MASQUE RÉUTILISABLE ALSICO Performance- WIEDERVERWENDBARE ALSICO MASKE Performance - REUSABLE ALSICO MASK Performance	 Alsico more than workwear	NL-FR-DE-EN Version 01 Date 14.05.2020
			pagina 1 of 1

TOEPASSING - APPLICATION - ANWENDUNG - APPLICATION

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98% Polyester / 2%Koolstofvezel - 98% Polyester/2% Fibre de carbon - 98% Polyester/2% Kohlefaser - 98% polyester /2% carbon fiber

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-Use the mask in situations where the wearing of this type of mask is recommended, for example when you go shopping, in public transportation, or in a work environment that doesn't require masks that comply to the medical device directive or regulation and neither to the PPE Regulation.

Het Masker dragen - Portez le masque correctement - Um die Maske korrekt zu tragen - To wear the mask correctly

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Using this mask constitutes by no means a reason not to follow or reduce all current rules established by health authorities like e.g. social distancing, hand and respiratory hygiene protocols, etc. This mask is no medical device and not in conformity with the medical device directive 93/42/EEC nor the regulation (EU)2017/745 for medical devices. This mask shall not be used in circumstances where masks compliant to this regulation are needed. This mask is no personal protection equipment and not in conformity with the PPE regulation (EU) 2016/425. This mask shall not be used in circumstances where masks compliant to this regulation are needed. This mask is not designed to prevent your lungs from airborne risks. This product is not suitable for children younger than 3 years due to choking hazard. Children wearing a mask should be supervised by an adult at all times. The mask doesn't allow heavy exertion like e.g. intensive sport. Wearing a mask will require adaptation. The wearer may feel to have less breathing air. This is normal and a sign the mask works. Do not use when breathing becomes difficult, dizziness or other distress occurs. Doctor's advice is recommended for people with respiratory diseases. Keep away from fire or heat source.

Het masker verwijderen - Retirez le masque correctement - Um die Maske korrekt zu entfernen - Remove the mask correctly

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Wash your hands with water and soap before touching the mask

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Don't touch the front side of the mask as it could be contaminated, only touch the ear loops or straps

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Remove the mask depending on the type without touching the front side

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Mask with ear loops Hold both ear loops and lift them gently to remove the mask

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Mask with ribbons First release the lowest tie and next the upper one, pull the mask away without touching the front side of the mask

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Mask with elastic straps first pull the lowest strap over your head and next the upper strap, without touching the mask on the front side

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Throw the mask in a separate and closable container, specifically used for this.

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The mask should be checked before (re)use on aging symptoms and that they are not damaged (no holes, no cracks, ...) or aging symptoms. If necessary, the mask should be replaced. The masks shall not be repaired. There is no direct link between the max number of washes that can be specified and the lifetime of the mask. The lifetime depends on the usage, maintenance, storage etc.

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The mask is reusable when correctly washed and dried. The mask can be washed min 25x. Used masks should be kept in a separate end closed container before washing. Always wash your hands with water and soap after opening and closing this container. Disinfect the container also after emptying it. Remove the stopper before washing and wash the mask separate.

Industriële wassen - Lavage industriel - Industrielle Wäscherei - Industrial laundering

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For industrial laundry processes we can use a lower temperature (min. 60 to 80°C) in combination with the right detergents and processes used for medical applications. Drying in tumbler at 85°C max.

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In order to have enough mechanical movement you should wash the masks together with a bath towel. Always use the same towel and only use it for this application.

Waszen voor gebruik en voor ieder hergebruik - Laver à chaque utilisation avant de réutiliser Waschen Sie die Maske vor jedem Gebrauch - Wash for each use and reuse

Kookwas met gewoon wasmiddel (min. 30 minuten) - Laver à 95°C avec détergent standard (min. 30 minutes) Waschen bei 95°C mit Standard-Waschmittel (mindestens 30 Minuten) -Wash at 95°C with standard detergent (min. 30 minutes)

Geen wasverzachter gebruiken - N'est pas utiliser d'adoucissant - Verwenden Sie keinen Weichspüler- Do not use softners

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When not used the masks should be stored in a dry place away from light, dirt or contamination. Used mask should stored for a short period in a closed separate container before being washed. Always disinfect the container after emptying it. At end of life of the mask, was hand dry it for the last time before ending it in for recycling or disposed in the household garbage.

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Alisco NV can not be held responsible for the consequences of incorrect use of the masks, nor for using it in situations where products are needed that need to comply either with the medical device directive and/or the PPE regulation. Use of the mask other than described above is at own risk Alisco NV is not responsible if the wear and tear is quicker than the max number of washes. Product warranty is excluded.

www.aliscomasks.com
masks@aliscomasks.com